

Restore Contact Layer Dressing with Silver



Protocol

This protocol is intended to supplement, not replace, your facility's guidelines and/or common practices.

Desired Outcomes

- To maintain a moist wound environment for the healing of partial to full-thickness wounds
- To help prevent pain during dressing removal
- To provide sustained antimicrobial activity

Equipment

1. Restore Contact Layer Dressing with Silver
2. Wound measuring guide
3. Restore Wound Cleanser or sterile saline for wound cleansing
4. Gauze or towel to dry periwound skin
5. Secondary cover dressing
6. Gloves

Procedures

- Preparation of wound area
- Application of wound dressing
- Removal of wound dressing
- Documentation of observations

Preparation of Wound Area

1. Assemble supplies and take to bedside.
2. Identify and inform patient of procedure.
3. Provide for privacy.
4. Wash hands and apply gloves.
5. Position patient so that the wound area is exposed.
6. Thoroughly rinse or irrigate the wound area with Restore Wound Cleanser or sterile saline, if appropriate. *(If necessary, the wound should be debrided).*
7. Clean and dry the periwound skin.
8. Measure the wound using the wound measuring guide.

Application of Dressing

1. Remove the protective clear tabs from both sides of the Restore Contact Layer Dressing with Silver.
Note: Restore Contact Layer Dressing with Silver tends to stick to latex gloves. Moisten latex gloves with normal sterile saline prior to use.
2. Apply Restore Contact Layer Dressing with Silver to cover the entire wound using aseptic technique. The dressing may overlap onto healthy skin.
3. Cover with appropriate secondary dressing using aseptic technique (gauze, transparent film, etc).
4. Secure secondary dressing with tape or other material.
5. Remove gloves and wash hands after completing procedure.

Removal of Dressing

1. Wash hands and put on gloves.
2. Remove secondary dressing.
3. Remove Restore Contact Layer Dressing with Silver
4. Irrigate wound base using Restore Wound Cleanser or sterile saline.
5. Reapply dressing if necessary.
6. Remove gloves and wash hands after completing procedure.

Note: Restore Contact Layer Dressing with Silver should be changed every 1 to 3 days. Dressing change frequency will depend on patient condition and the healing progression.

Documentation of Observations

Document dressing changes, wound measurements and other pertinent observations regarding the wound or periwound site.

Assess

- Key observations to document with wound dressing change:
- Size of wound
- Depth of tissue involvement (partial-thickness, full-thickness, stage)
- Presence of undermining or tunnelling
- Anatomic location
- Wound Base (granulation, epithelialization, subcutaneous tissue, muscle, bone or other underlying structures, non-viable/necrotic tissue, color, exudate amount/odor/color)
- Edge of wound
- Presence of foreign bodies
- Condition of surrounding skin

Plan

Select interventions to provide appropriate local wound care:

- Prevent and manage infection
- Pain management
- Cleanse wound
- Remove nonviable tissue
- Manage exudate
- Eliminate dead space
- Control Odor
- Protect wound and periwound skin
- Maintain a moist wound environment

Implement

Follow Protocol for the Restore Contact Layer Dressing with Silver

- Reduce or eliminate factors that may interfere with wound healing
- Provide systemic support for wound healing

Evaluate

- Reassess wound with each dressing change.
- Frequency of dressing changes should be dictated by facility protocol.
- If no progress can be demonstrated within two to four weeks, reevaluate the overall treatment plan, as well as adherence to this plan, making modifications as necessary.

Restore Contact Layer Dressing with Silver Advantages

- Provides sustained antimicrobial activity for up to seven days
- Maintains a moist wound environment
- Helps reduce or eliminate pain at dressing changes
- Helps minimize damage to newly formed tissue

Ordering Information

| Restore Contact Layer, Silver | Stock No | Unit of Sale |
|-------------------------------|----------|--------------|
| 4" x 5" (10 cm x 12.7 cm) | 509341 | 10 |
| 6" x 8" (15 cm x 20 cm) | 509342 | 10 |

Restore Wound Cleanser

| | | |
|---|--------|--------|
| 8 oz. (236 mL) Trigger Spray Bottle, Non-Sterile | 529975 | 12/box |
| 12 oz. (354 mL) Trigger Spray Bottle, Non-Sterile | 529976 | 12/box |

See Instructions for Use for important information regarding the use of this product at www.hollisterwoundcare.com/products/ifus.html.

Contraindications

Restore Contact Layer with Silver should not be used on individuals who are sensitive to silver or who have had an allergic reaction to the dressing or one of its components.

* **Caution:** Federal law restricts this device for sale or on the order of a physician or licensed healthcare professional.



Manufactured for
Hollister Wound Care LLC
1580 South Milwaukee Avenue
Suite 405
Libertyville, Illinois 60048
1.888.740.8999

Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400

www.hollisterwoundcare.com