









Suspected Deep Tissue Injury (DTI):	Shallow Pressure Ulcer:	Deep Pressure Ulcer:	Unstageable Pressure Ulcer:	Venous Ulcer:	Arterial Wound:	Diabetic Ulcer:	Skin Tear:
<p>Purple or maroon localized area of discolored intact skin or blood-filled blister</p>  <p>Restore Hydrocolloid Extra Thin Dressing to protect from friction/shear</p>	<p>Superficial wound appearing as an abrasion, blister or partial-thickness skin loss</p>  <p>Dry: Restore Hydrogel Dressing Wet: Restore Hydrocolloid Dressings (light to moderate exudate) or Restore Foam Dressing (moderate exudate)</p>	<p>Full-thickness tissue loss</p>  <p>Dry: Restore Hydrogel Dressing Wet: Restore Foam Dressing; Use with Restore Calcium Alginate for additional absorbency</p>	<p>Full-thickness tissue loss in which the base of the ulcer is covered by slough or eschar</p>  <p>Restore Hydrogel Dressing until debrided, then re-evaluate once able to stage the wound</p>	<p>Edema and ulceration to lower leg caused by venous hypertension. These wounds are typically shallow with irregular shape</p>  <p>Dry: Restore Contact Layer with compression bandage Wet: Restore Foam Dressing with compression; Use with Restore Calcium Alginate for additional absorbency</p>	<p>Ulcer due to partial or complete occlusion of the vessel</p>  <p>Restore Contact Layer or Restore Hydrogel Dressing</p>	<p>Caused by repetitive stress, pressure or trauma to the insensate foot</p>  <p>Dry: Restore Hydrogel Dressing Dry to Moist: Restore Contact Layer</p>	<p>A break in the skin due to friction, shear or trauma</p>  <p>Restore Contact Layer</p>
<p>Special Note: Monitor status of skin closely, as it can progress quickly into an open wound that may require a different dressing option</p>	<p>Antimicrobial Option: Restore Contact Layer, Silver* (light exudate) or Restore Foam Dressing, Silver* (moderate exudate)</p>	<p>Antimicrobial Option: Restore Foam Dressing, Silver* or Restore Calcium Alginate, Silver*</p>	<p>Antimicrobial Option: Restore Contact Layer, Silver*</p>	<p>Antimicrobial Option: Restore Contact Layer, Silver*, Restore Foam Dressing, Silver* or Restore Calcium Alginate, Silver*</p>	<p>Antimicrobial Option: Restore Contact Layer, Silver*</p>	<p>Antimicrobial Option: Restore Contact Layer, Silver* or Restore Calcium Alginate, Silver*</p>	<p>Antimicrobial Option: Restore Contact Layer, Silver*</p>

Pressure Ulcer Staging

Deep Tissue Injury (DTI):

Purple or maroon localized area of discolored intact skin or blood-filled blister due to damage of underlying soft tissue from pressure and/or shear. The area may be preceded by tissue that is painful, firm, mushy, boggy, warmer or cooler as compared to adjacent tissue.

Stage I:

Intact skin with non-blanchable redness of a localized area usually over a bony prominence. Darkly pigmented skin may not have visible blanching; its color may differ from the surrounding area.

Stage II:

Partial-thickness loss of dermis presenting as a shallow open ulcer with a red-pink wound bed without slough. May also present as an intact or open/ruptured serum-filled blister.

Stage III:

Full-thickness tissue loss. Subcutaneous fat may be visible but bone, tendon or muscle are not exposed. Slough may be present but does not obscure the depth of tissue loss. May include undermining and tunneling.

Stage IV:

Full-thickness tissue loss with exposed bone, tendon or muscle. Slough or eschar may be present on some parts of the wound bed. Often include undermining or tunneling.

Unstageable:

Full-thickness tissue loss in which the base of the ulcer is covered by slough (yellow, tan, gray, green, or brown) and/or eschar (tan, brown or black) in the wound bed.

Source: National Pressure Ulcer Advisory Panel (NPUAP), 2007



An alliance of Hollister Incorporated and Laboratoires URGO

Hollister Wound Care Products



Restore Calcium Alginate Dressing	
529938	2" x 2"
529937	4" x 4"
529939	4" x 8"
529940	12" rope



Restore Calcium Alginate Dressing, Silver*	
529967	2" x 2"
529968	4" x 4.75"
529969	1" x 12" rope



Restore Hydrocolloid Dressing	
519953	4" x 4" without tapered edges
519954	6" x 8" without tapered edges
519955	8" x 8" without tapered edges
519956	4" x 4" with tapered edges
519963	6" x 6" with tapered edges
519957	6" x 8" with tapered edges
519958	8" x 8" with tapered edges
519959	triangle 17 sq. in. with tapered edges (sacral)
519965	triangle 26.5 sq. in. with tapered edges (sacral)

Restore Extra Thin Hydrocolloid Dressing	
519921	4" x 4"
519923	6" x 8"
519925	8" x 8"

Restore Hydrocolloid Dressing with Foam Backing	
519930	4" x 4"
519932	6" x 8"
519935	8" x 8"



Restore Hydrogel Dressing	
529974	3 oz. tube of gel (non-sterile)
529972	4"x 4" gauze sponge
529973	2" x 3.5 yd gauze packing strip

Hollister Wound Care Products with Triact Technology



Restore Contact Layer Dressing with TRIACT Technology	
509338	4" x 5"
509339	6" x 8"




Restore Contact Layer Dressing, Silver* with TRIACT Technology	
509341	4" x 5"
509342	6" x 8"



Restore Foam Dressing, Adhesive with TRIACT Technology	
509347	4" x 4" (1.5" x 1.5" foam pad)
509348	6" x 6" (4" x 4" foam pad)
509349	6" x 8" (3.5" x 5.6" foam pad)



Restore Foam Dressing, Non-Adhesive with TRIACT Technology	
509343	4" x 4"
509344	6" x 8"

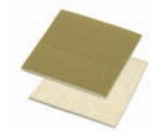

Restore Foam Dressing, Silver,* Non-Adhesive with TRIACT Technology	
509345	4" x 4"
509346	6" x 8"



*For products containing silver

Note: Rx Only.

Caution: Federal laws restricts this device to sale or on the order of a physician or licensed healthcare professional.

Contraindications – Known sensitization to silver and/or other dressing components.

Complete Instructions for Use for all Hollister Wound Care products are available online at www.hollisterwoundcare.com or by calling customer care at 1.888.740.8999.



An alliance of Hollister Incorporated and Laboratoires URGO

Manufactured for
Hollister Wound Care LLC
 1580 South Milwaukee Avenue
 Suite 405
 Libertyville, IL 60048
 1.888.740.8999


Distributed in Canada by
Hollister Limited
 95 Mary Street
 Aurora, Ontario L4G 1G3
 1.800.263.7400

www.hollisterwoundcare.com

Hollisterwoundcare and wave logo are trademarks of Hollister Incorporated. Restore, TRIACT and graphic are trademarks of Hollister Wound Care LLC. Covered under one or more of the following patents: U.S. Patent No. 6,270,792, 6,794,555, and 6,080,420. Made and printed in USA. ©2008 Hollister Wound Care LLC. 910299-408

Hollister Wound Care Dressing Selection Guide

Comfort, Care and Dignity

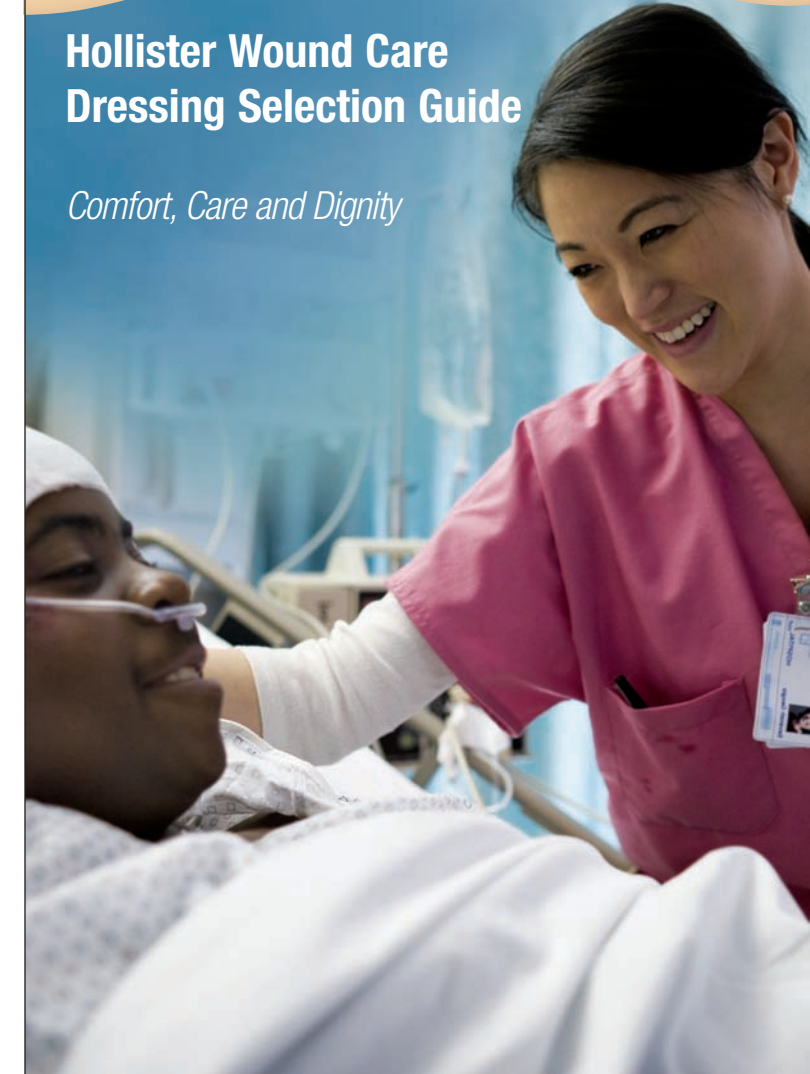

An alliance of Hollister Incorporated and Laboratoires URGO