For external use only.

DESCRIPTION

Restore Hydro-Shield Foam Dressing with border is a sterile wound dressing consisting of three layers:

- a soft, conformable microporous absorbent polyurethane foam pad.
- a breathable, waterproof, low friction polyurethane membrane backing.
- a pressure sensitive acrylic adhesive.

The product is applied to the skin with the central part of the dressing in contact with the wound. The Restore Hydro-Shield Foam Dressing with border is self adhesive and therefore no secondary dressing is required. The membrane is a waterproof layer that provides a bacterial barrier. Restore Hydro-Shield Foam Dressing with border is self adhesive, soft, highly absorbent, conformable and provides a moist healing wound environment.

Restore Hydro-Shield Foam Dressing with border is supplied in sterile, individual packaging for single use.

INDICATIONS FOR USE

Restore Hydro-Shield Foam Dressing with border may be used throughout the healing process and is indicated for moderate to heavily exuding chronic and acute wounds including:

Chronic wounds:

- Pressure ulcers
- Leg ulcers of mixed etiology
- Venous ulcers
- Arterial ulcers
- Diabetic ulcers

Acute wounds:

- Lacerations
- Abrasions
- Graft wounds and donor sites
- Post-operative surgical wounds
- Trauma wounds
- Partial thickness burns

Restore Hydro-Shield Foam Dressing with border is suitable for use under compression bandages/wraps due to its ability to retain exudate.

INSTRUCTIONS FOR USE

1. Wound Preparation:

- Cleanse the wound in accordance with established procedures.
- Carefully dry the periwound tissue.

2. Dressing Application:

- Select the appropriate dressing size to ensure the central foam pad will extend beyond the open wound at least ¾" [2cm] in all directions.
- Remove the release liners from the dressing, and avoid touching the pad that will go directly onto the wound.
- Center the dressing on the wound and apply it gently to the wound site. Press the dressing onto the surrounding skin by gentle application of pressure.
- The dressing can be used under compression.

3. Dressing Change and Removal:

- Restore Hydro-Shield Foam Dressing with border can be left in place for up to 7 days, depending on the level of exudate and the clinical condition of the wound.
- Duration of treatment is determined by the physician and depends on the wound type and condition.
- Remove the dressing by peeling off from a corner using gentle pressure.
- Gently remove the dressing from the wound bed and discard.

PRECAUTIONS FOR USE

Do not use on patients with a known sensitivity to polyurethane membranes, foams or acrylic adhesives.

Do not use with oxidizing solutions such as hypochlorite or hydrogen peroxide.

CONTRAINDICATIONS

Restore Hydro-Shield Foam Dressing with border is not indicated for surgical implantation.

HOW SUPPLIED

Restore Hydro-Shield Foam Dressing with border is supplied in 3 sizes, and packaged in boxes of 10 dressings.

- REF. 529304:** 4" x 4" (10 cm x 10 cm)
with pad size 2" x 2" (5 cm x 5 cm)
- 529305:** 6" x 6" (15 cm x 15 cm)
with pad size 4.5" x 4.5" (11.4 cm x 11.4 cm)
- 529306:** 8" x 8" (20 cm x 20 cm)
with pad size 5.4" x 5.4" (13.7 cm x 13.7 cm)

Each dressing is individually packed in a sterile pouch. Do not re-sterilize.

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©2010 Hollister Incorporated.
Hollisterwoundcare and wave logo are trademarks of Hollister Incorporated.
Restore, Hydro-Shield and graphic are trademarks of Hollister Wound Care, LLC.