For external use only.

DESCRIPTION

Restore Hydro-Shield Foam Dressing without border is a sterile wound dressing consisting of two layers:

- a soft, conformable microporous absorbent polyurethane foam pad.
- a breathable, waterproof, low friction polyurethane membrane backing.

The membrane is a waterproof layer that provides a bacterial barrier. Restore Hydro-Shield Foam Dressing without border is soft, highly absorbent, conformable and provides a moist wound healing environment.

Restore Hydro-Shield Foam Dressing without border is supplied in sterile, individual packaging for single use.

INDICATIONS FOR USE

Restore Hydro-Shield Foam Dressing without border may be used throughout the healing process and is indicated for moderate to heavily exuding chronic and acute wounds including:

Chronic wounds:

- Pressure ulcers
- Leg ulcers of mixed etiology
- Venous ulcers
- Arterial ulcers
- Diabetic ulcers

Acute wounds:

- Lacerations
- Abrasions
- Graft wounds and donor sites
- Post-operative surgical wounds
- Trauma wounds
- Partial thickness burns

Restore Hydro-Shield Foam Dressing without border is suitable for use under compression bandages/wraps due to its ability to retain exudate. Restore Hydro-Shield Foam Dressing without border is recommended in the treatment of wounds in which the surrounding skin is fragile, friable or damaged.

INSTRUCTIONS FOR USE

1. Wound Preparation:

- Cleanse the wound in accordance with established procedures.
- Carefully dry the periwound tissue.

2. Dressing Application:

- Select the appropriate dressing size to ensure the dressing will extend beyond the open wound at least ¾" [2cm] in all directions.
- The Restore Hydro-Shield Foam Dressing without border can be cut as needed.
- Apply to the wound with the white side down in contact with the wound.
- Secure the dressing using a bandage or tape.
- The dressing can be used under compression.

3. Dressing Change and Removal:

- Restore Hydro-Shield Foam Dressing without border can be left in place for up to 7 days, depending on the level of exudate and the clinical condition of the wound.
- Duration of treatment is determined by the physician and depends on the wound type and condition.
- Gently remove the dressing from the wound bed and discard.

PRECAUTIONS FOR USE

Do not use on patients with a known sensitivity to polyurethane membranes or foams.

Do not use with oxidizing solutions such as hypochlorite or hydrogen peroxide.

CONTRAINDICATIONS

Restore Hydro-Shield Foam Dressing without border is not indicated for surgical implantation.

HOW SUPPLIED

Restore Hydro-Shield Foam Dressing without border is supplied in 4 sizes, packaged in boxes of 10 dressings:

REF. 529300: 3" x 3" (7.6 cm x 7.6 cm)

529301: 4" x 4" (10 cm x 10 cm)

529302: 6" x 6" (15 cm x 15 cm)

529303: 8" x 8" (20 cm x 20 cm)

Each dressing is individually packed in a sterile pouch. Do not re-sterilize.

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Hollisterwoundcare and wave logo are trademarks of Hollister Incorporated.
Restore, Hydro-Shield and graphic are trademarks of Hollister Wound Care, LLC.

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©2010 Hollister Incorporated.