Calcium Alginate Dressing is indicated for external use only.

Product Description

Calcium Alginate Dressing is a sterile, non woven pad composed of a high G (guluronic acid) calcium alginate, and carboxymethylcellulose (CMC). As wound fluid is absorbed, the alginate forms a gel, which assists in maintaining a moist environment for optimal wound healing, and allows intact removal.

Calcium Alginate Dressing protects the wound and aids autolytic debridement therefore facilitating wound healing.

Indications for Use

For the management of moderate to heavily exuding, partial- to full-thickness wounds including:

- Pressure Ulcers (Stages I-IV)
- Arterial Ulcers
- Venous Stasis Ulcers
- Post-Surgical Incisions
- Donor Sites
- Trauma Wounds (dermal lesions, trauma injuries or incisions)
- Diabetic Ulcers
- Dermal Lesions
- 1st and 2nd Degree Burns

This product can also be used under compression bandages.

As the product contains alginate, it may assist in supporting the control of minor bleeding in superficial wounds.

Contraindications

Calcium Alginate Dressing is not indicated for use on the following:

- third degree burns
- individuals with a known sensitivity to alginates
- surgical implantation
- to control heavy bleeding
- for direct application on dry or lightly exuding wounds.

Precautions

The dressing may adhere if used on dry or very lightly exuding wounds. If the dressing is not easily removed, moisten it with sterile saline solution prior to removal.

The dressing performance may be impaired by excess use of petroleum-based ointments.

DIRECTIONS FOR USE

Site Preparation

- Debride when necessary and irrigate the wound site in accordance with standard protocols.
- Remove excess solution from surrounding skin.

Dressing Selection

- Select a size of Calcium Alginate Dressing that is slightly larger than the wound.

Dressing Application

1. Cut (using clean scissors) or fold the dressing to fit the wound. Loosely fill deep wounds, ensuring the dressing does not overlap the wound margins.
2. Apply to wound bed directly.
3. Cover and secure Calcium Alginate Dressing with a non-occlusive secondary dressing.

Dressing Change and Removal

1. Dressing change frequency will depend on wound condition and the level of exudate. Initially it may be necessary to change the dressing every 24 hours.
2. Reapply Calcium Alginate Dressing when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.
3. Gently remove the secondary dressing.
4. If the wound appears dry, saturate the dressing with sterile saline solution prior to removal.
5. Gently remove the dressing from the wound bed and discard.
6. Irrigate the wound site in accordance with standard protocols prior to application of a new dressing.

Storage

Store at ambient temperature (below 25°C/77 °F).

Do not re-sterilize.

The use by date of this product is printed on the packaging.

Calcium Alginate Dressing
Pansement d'alginate de calcium
Apósito de alginato de calcio

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Hollisterwoundcare and wave logo are trademarks of Hollister Incorporated.
 Restore is a trademark of Hollister Wound Care, LLC.

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