Calcium Alginate Dressing, Silver is indicated for external use only.

Product Description

Calcium Alginate Dressing, Silver is a sterile, non woven pad composed of a high G (guluronic acid) calcium alginate, carboxymethylcellulose (CMC) and ionic silver complex (Silver Sodium Hydrogen Zirconium Phosphate), which releases silver ions in the presence of wound fluid. As wound fluid is absorbed the alginate forms a gel, which assists in maintaining a moist environment for optimal wound healing, and allows intact removal.

The silver ions released in the presence of wound exudate protect the dressing from a broad spectrum of microorganisms over a period of up to fourteen (14 days), based on in-vitro testing. Odor reduction results from the antibacterial effect in the dressing. Calcium Alginate Dressing, Silver is an effective barrier to bacterial penetration.

Calcium Alginate Dressing, Silver protects the wound and aids autolytic debridement therefore facilitating wound healing.

Indications for Use

Calcium Alginate Dressing, Silver is indicated for moderate to heavily exuding partial to full thickness wounds including:

- post-operative wounds
- trauma wounds (dermal lesions, trauma injuries or incisions)
- leg ulcers
- pressure ulcers
- diabetic ulcers
- graft and donor sites
- 1st and 2nd degree burns
- post-operative surgical wounds

This product can also be used under compression bandages.

As the product contains alginate, it may assist in supporting the control of minor bleeding in superficial wounds.

Calcium Alginate Dressing, Silver is suitable for use, under medical supervision, on wounds that are critically colonized.

Contraindications

Calcium Alginate Dressing, Silver is not indicated for use on the following:

- third degree burns
- individuals with a known sensitivity to alginates or silver
- surgical implantation
- to control heavy bleeding
- for direct application on dry or lightly exuding wounds.

Precautions

Systemic antimicrobial therapy should be considered when wound infection is evident. Calcium Alginate Dressing, Silver may be used, under medical supervision, in conjunction with systemic antibiotics.

The dressing may adhere if used on dry or very lightly exuding wounds. If the dressing is not easily removed, moisten it with sterile saline solution prior to removal.

The dressing performance may be impaired by excess use of petroleum-based ointments.

Caution: Federal [US] Law restricts this device to sale by or on the order of a physician [or properly licensed practitioner].

DIRECTIONS FOR USE

Site Preparation

- Debride when necessary and irrigate the wound site in accordance with standard protocols.
- Remove excess solution from surrounding skin.

Dressing Selection

- Select a size of Calcium Alginate Dressing, Silver that is slightly larger than the wound.

Dressing Application

1. Cut (using clean scissors) or fold the dressing to fit the wound. Loosely fill deep wounds, ensuring the dressing does not overlap the wound margins.
2. Apply to wound bed directly.
3. Cover and secure Calcium Alginate Dressing, Silver with a nonocclusive secondary dressing.

Dressing Change and Removal

1. Dressing change frequency will depend on wound condition and the level of exudate. Initially it may be necessary to change the dressing every 24 hours.
2. Reapply Calcium Alginate Dressing, Silver when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.
3. Gently remove the secondary dressing.
4. If the wound appears dry, saturate the dressing with sterile saline solution prior to removal.
5. Gently remove the dressing from the wound bed and discard.
6. Irrigate the wound site in accordance with standard protocols prior to application of a new dressing.

Storage

Store at ambient temperature (below 25°C/77 °F).

Do not re-sterilize.

The use by date of this product is printed on the packaging.

Apósito de alginato de calcio con
argenteo
Pansement d'alginate de calcium,
Argenté
Calcium Alginate Dressing,
Silver

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Hollisterwoundcare and wave logo are trademarks of Hollister Incorporated.
Restore is a trademark of Hollister Wound Care, LLC.

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