ESTERIL
STÉRILE
STERILE
Rx Only

de plata-sodio-hidrógeno-zirconio-fosfato
Apósito contiene nominalmente 5%, por peso,
Apósito de Alginato de Calcio, Plata
en poids de phosphate de zirconium-hydrogène-sodium-argent
Pansement contient essentiellement 5%
Pansement d'alginat de calcium, Argent
Silver Sodium Hydrogen Zirconium
Dressing Contains nominally 5%, by weight,
Calcium Alginate Dressing, Silver
INSTRUCTIONS/MODE D'EMPLOI/INSTRUCCIONES

Restore

Graphical Symbols

Symboles graphiques

Simbolos Gráficos

Store at ambient temperature
Rangez à température ambiante
Guarde a temperatura ambiente

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hollisterwoundcare and wave logo are trademarks of Hollister Incorporated.
Restore is a trademark of Hollister Wound Care, LLC.

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Calcium Alginate Dressing, Silver

Calcium Alginate Dressing, Silver is a sterile, non woven pad composed of a high G (guluronic acid) calcium alginate, carboxymethylcellulose (CMC) and ionic silver complex (Silver Sodium Hydrogen Zirconium Phosphate), which releases silver ions in the presence of wound fluid.

As wound fluid is absorbed the alginate forms a gel, which assists in maintaining a moist environment for optimal wound healing, and allows intact removal.

The silver ions released in the presence of wound exudate are an effective antimicrobial agent against a broad spectrum of microorganisms frequently associated with the bacterial colonization and infection of wounds.

Odor reduction results from the antibacterial effect in the dressing. Calcium Alginate Dressing, Silver is an effective barrier to bacterial penetration.

Indications for Use

Calcium Alginate Dressing, Silver is indicated for:

- moderate to heavily exuding partial to full thickness wounds including
- post-operative wounds
- trauma wounds
- leg ulcers
- pressure ulcers
- diabetic ulcers
- graft and donor sites

Calcium Alginate Dressing, Silver is indicated for external use only.

Contra indications

Calcium Alginate Dressing, Silver is not indicated for use on the following:

- third-degree burns
- dry or lightly exuding wounds
- individuals with a known sensitivity to alginates or silver
- to control heavy bleeding
- surgical implantation

Precautions

Systemic antimicrobial therapy should be considered when wound infection is evident. Calcium Alginate Dressing, Silver may be used, under medical supervision, in conjunction with systemic antibiotics

The dressing may adhere if used on dry or very lightly exuding wounds. If the dressing is not easily removed, moisten it with sterile saline solution prior to removal
The dressing performance may be impaired by excess use of petroleum-based ointments.

Directions for use

Site Preparation

- Debride when necessary and irrigate the wound site in accordance with standard protocols
- Remove excess solution from surrounding skin

Dressing Selection

- Select a size of Calcium Alginate Dressing, Silver that is slightly larger than the wound

Dressing Application

1. Cut (using clean scissors) or fold the dressing to fit the wound. Loosely fill deep wounds, ensuring the dressing does not overlap the wound margins.
2. Apply to wound bed directly.
3. Cover and secure Calcium Alginate Dressing, Silver with a nonocclusive secondary dressing.

Dressing Change and Removal

1. Dressing change frequency will depend on wound condition and the level of exudate. Initially it may be necessary to change the dressing every 24 hours.

2. Reapply Calcium Alginate Dressing, Silver when the secondary dressing has reached its absorbent capacity or whenever good wound care practice dictates that the dressing should be changed.
3. Gently remove the secondary dressing.
4. If the wound appears dry, saturate the dressing with sterile saline solution prior to removal.
5. Gently remove the dressing from the wound bed and discard.
6. Irrigate the wound site in accordance with standard protocols prior to application of a new dressing.

Storage

Store at ambient temperature.

Do not re-use.

Do not re-sterilize.

Do not use if primary package is damaged/opened.

The use by date of this product is printed on the packaging.

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