ESTÉRIL  
STERILE  
STERILE

Apósito Hidrocolloide Extra Delgado  
Pansement hydrocolloïde Ultra-Mince  
Extra Thin Hydrocolloid Dressing

INSTRUCCIONES/MODE D'EMPLOI/INSTRUCCIONES

**Restore**

Graphical Symbols  
Symboles graphiques  
Símbolos Gráficos

USA: 1-800-323-4060/FAX Order: 847-680-1017  
CANADA: 1-800-263-7400/FAX Order: 1-800-432-8846  
ETATS-UNIS: 1-800-323-4060  
Commande par télécopieur: 847-680-1017  
CANADA: 1-800-263-7400  
Commande par télécopieur: 1-800-432-8846  
[www.hollisterwoundcare.com](http://www.hollisterwoundcare.com)

hollisterwoundcare and wave logo are trademarks of Hollister Incorporated. Restore is a trademark of Hollister Wound Care, LLC.

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## Extra Thin Hydrocolloid Dressing

The Extra Thin Hydrocolloid Dressing is a sterile, occlusive dressing. The flexible outer layer helps isolate the wound against bacterial and viral (HIV-1 and HBV) contaminants and other external contamination such as urine and feces while the dressing remains intact without leakage.<sup>†</sup> The self-adhesive dressing helps maintain a moist environment for wound healing.

### Indications for Use

- To protect skin from friction injury
- To manage superficial wounds with minimal or no exudate

### Precautions/Observations

Extra Thin Hydrocolloid dressings have been shown in viral penetration laboratory experiments to act as a barrier to bacteria and viruses (Hepatitis B and C and Human Immunodeficiency Virus HIV) while the dressings remain intact without leakage.<sup>†</sup>

### Preparing the Wound Area

Before applying a dressing, the wound area should be thoroughly rinsed or irrigated with an appropriate solution. The skin should be clean, dry and free from oily residue to ensure secure application.

### Applying the Dressing

To ensure attachment to healthy skin, the dressing should extend at least one inch beyond the wound edge. Dressings may be overlapped or cut to accommodate the wound site.

1. Remove the printed release paper from the patient side of the dressing.
2. Center the dressing over the wound site. Press the dressing to the skin and smooth it to remove all wrinkles.

### Wear Time

The dressing should be left in place until one or more of the following occurs:

- leakage of exudate
- loosening of the edges of the dressing
- tenderness or signs of infection
- seven days have elapsed
- there is no longer a clinical need for the dressing

If the dressing has achieved a seven-day wear time, and the area needs continued treatment, apply a new dressing.

### Removing the Dressing

3. Carefully lift an edge of the dressing and peel away from the skin.




### Contraindications

The dressing should not be used on patients with active vasculitis or ulcers involving muscle, tendon or bone; on patients with deep systemic infections; or on patients with signs of active local infection at the wound site (erythema, cellulitis or purulent discharge).

**Note:** The dressing should be left in place (not more than 7 days).

<sup>†</sup> The use of this dressing neither guarantees nor warranties against the transmission of the HIV-1 and HBV virus.

Reference: ASTM F1671-97 standard test method for resistance of materials used in protective clothing to penetration by blood-borne pathogens using Phi-X174 Bacteriophage as a test system.

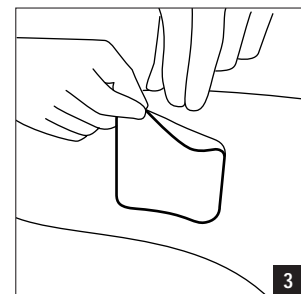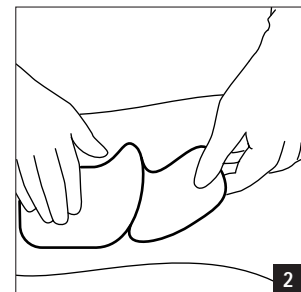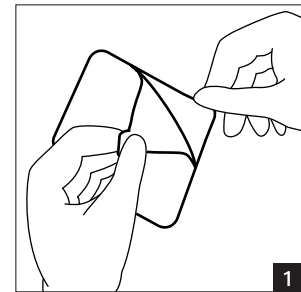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