ESTÉRILE  
 STÉRILE  
 STÉRILE

Apósito Absorbente de Olores

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hollisterwoundcare and wave logo are trademarks of Hollister Incorporated.  
 Restore is a trademark of Hollister Wound Care, LLC.

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Manufactured for:  
**Hollister Wound Care, LLC**  
 Libertyville, Illinois 60048

Distributed in Canada by:  
 Distribué au Canada par:

**Hollister Limited**  
 Aurora, Ontario L4G 1G3

Printed in USA.  
 Imprimé aux États-Unis.  
 Impreso en EE.UU.  
 © 2006 Hollister Incorporated.  
 907689-A407

# Odor-Absorbent Dressing

## Indications for Use

Odor-Absorbent Dressing effectively eliminates offending odors resulting from infection or bacterial contamination in surgical, traumatic, cancerous or gangrenous wounds, pressure ulcers, or venous stasis ulcers. The dressing is composed of a foam matrix impregnated with carbon between two layers of soft, non-woven fabric and conforms to the contours of the patient's body.

Odor-Absorbent Dressing is primarily intended as a secondary dressing but may be used as a primary dressing over a **non-exuding** wound. For particularly malodorous wounds, two or more Odor-Absorbent Dressings may be applied in layers over the same wound. For wounds that exceed dimensions of the Odor-Absorbent Dressing, two or more dressings may be overlapped and taped together.

The dressing(s) must cover the entire wound or primary dressing with a slight overlap on all sides. To perform effectively, the dressing must be taped around its perimeter, thus forcing odor molecules into the dressing and not under it.

## Recommended Change Frequency

Studies have shown the Odor-Absorbent Dressing to be continuously effective over a period of several days; these projections are **approximations**. The actual change frequency will vary, depending on the rate and nature of exudation and odor.

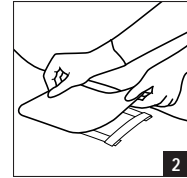
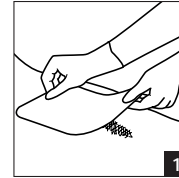
**Note:** To maintain sterility and maximum odor-absorbent capacity, store unopened dressings in the peel-pouch.

## Applying the Dressing

Peel open the envelope; remove the Odor-Absorbent Dressing.

1. If using the Odor-Absorbent Dressing as a **primary** dressing over a non-exuding wound, place it directly over the prepared wound surface.
2. If it is being used as a secondary dressing, apply it over the contact layer.

After placement of the dressing(s), complete application by taping around the perimeter.

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